


TAP TURNER


(ADJUSTABLE)

PURPOSE:


To lessen the strain on fingers and wrist in turning tap; to give leverage for weak hand.


STANDARD:

Suitable for 4-pronged  taps.

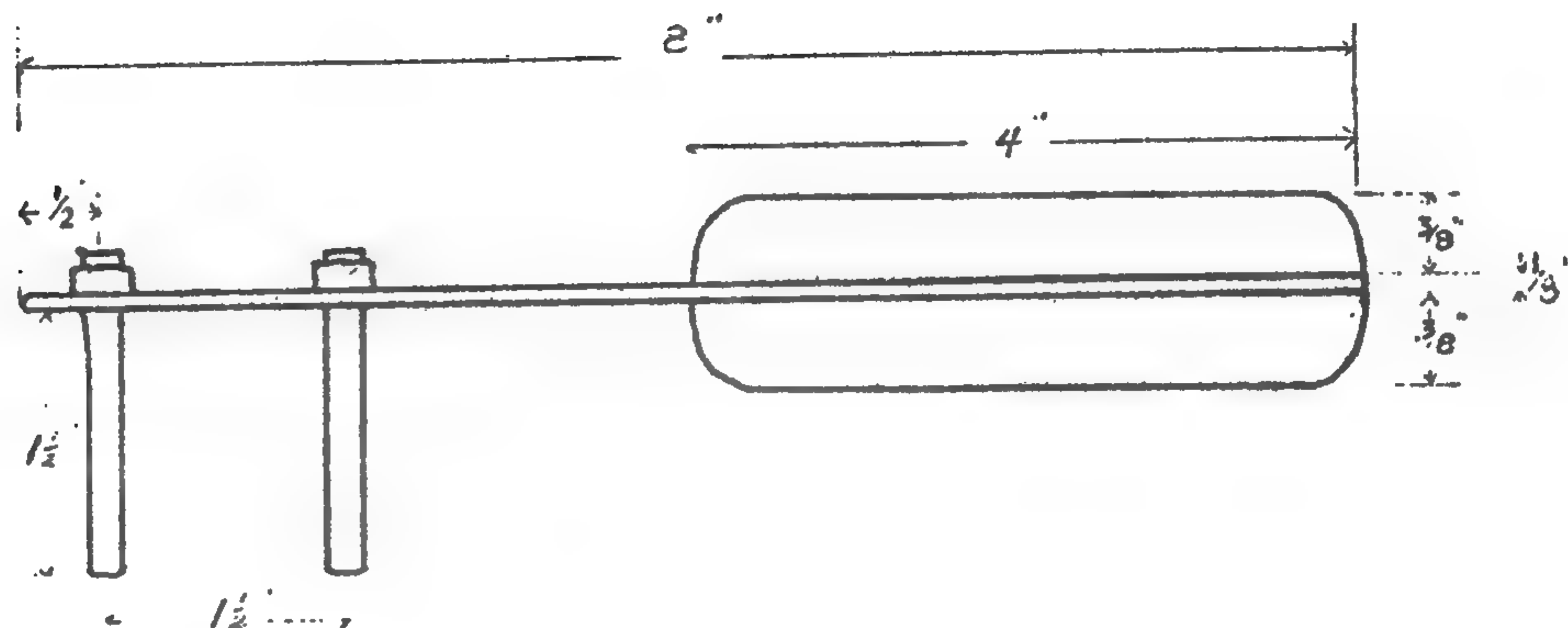

ADJUSTABLE:

Better for 2-pronged  taps. May be fitted to tap, then tightened with pliers.

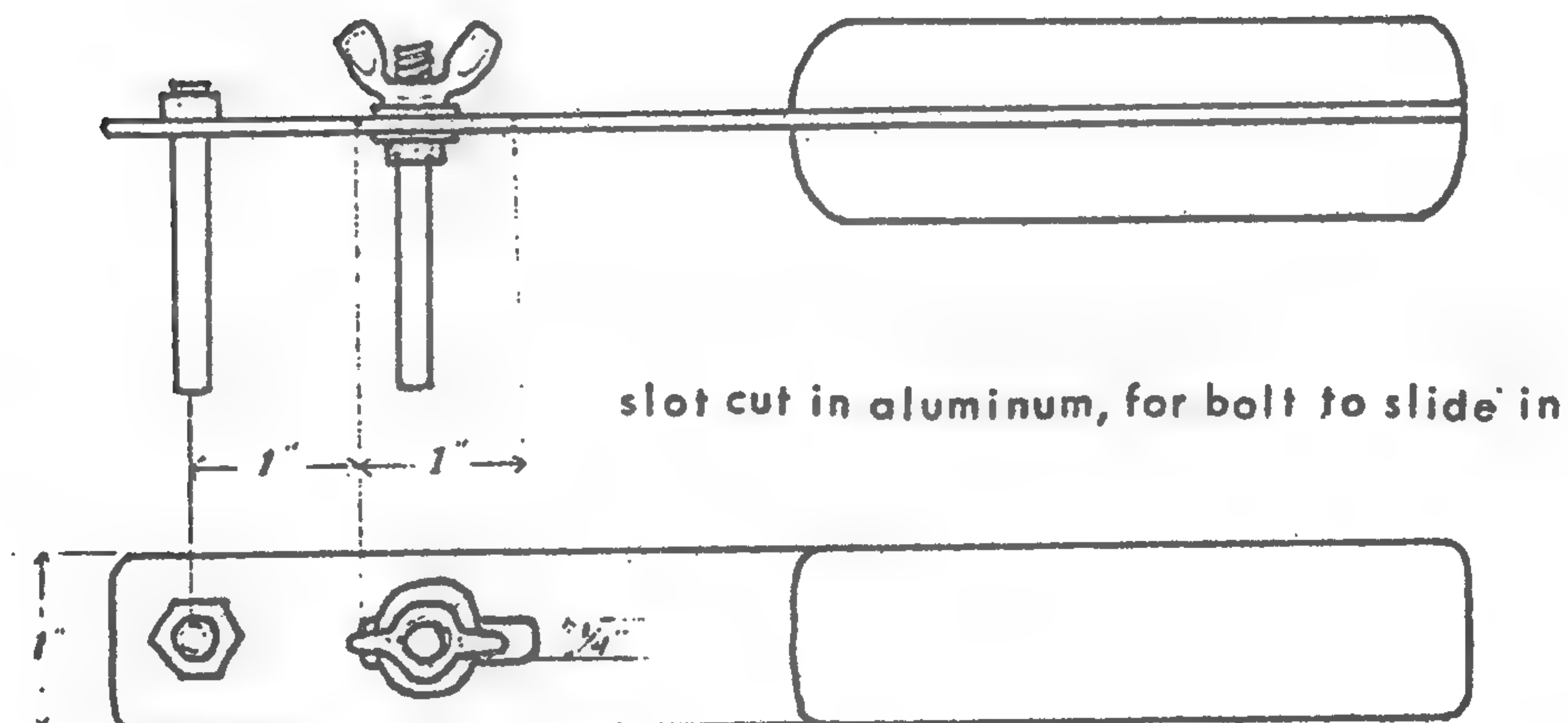

SIDE

TOP

MATERIALS:

8" of 1/8" x 1" aluminum 65 ST 6 bar (firm).

2 of 1/2" x 2" carriage bolts,
(head sawn off, with only last 1/2" threaded;
3/4" threaded for one if for adjustable type.)

2 of 1/2" nuts.

2 of 3/8" x 1" wood, 4" long.

1 of 1/2" wing nut)
2 of 1/2" washers) For adjustable type.

CANADIAN ARTHRITIS SOCIETY